**Description:** PSD - 6500 - Booklet UI

FG Stock No.: n/a

Pkg Spec No.: 34-8713-3512-0    Pkg Structure No.: SS-44131

Barcode No: n/a    Size/BWR: n/a

Dimensions: 105mm x 148mm / 4.13" x 5.82"

Folded Size: XXXmm x XXXmm / xx" x xx"

File name: O-AI14-MAC10-34-8713-3512-0.ai

**Job No:** 159100

**Rev. Date:** 28/11/2013

**Operator:** 1

**Process:** n/a

**Version**    **Regions**    **Itarus**

**File No:** 74913

**Rev. Time:** 16:30

**Asset ID:** 9139

**UIC**  
**rywsam**

Itarus House,  
Tenter Road,  
Moulton Park,  
Northampton  
NN3 6PZ

Need help? Contact us:

**+44 (0)1604 468100**

mail@itarus.com  
www.itarus.com

**Colours:** 1

Black











TECHNICAL  
DRAWING

PrintSpec™

(GB) (IE) (ZA)	3M™ Reusable Half Mask	3-5	(GR)	3M™ Επαναχρησιμοποιούμενη Μάσκα Μισού Προσώπου	43-46	(RO)	3M™ Semimasca Reutilizabila	77-80
(FR) (CH) (BE)	3M™ Demi-Masque réutilisable	6-9	(PL)	3M™ Polmaska wielokrotnego użytku	47-50	(RU) (BY) (AZ)	3M™ Полумаска для многоразового использования	81-84
(DE) (CH) (AT)	3M™ Wiederverwendbare Halbmaske	10-13	(HU)	3M™ többször használható légzésvédő félálarc	51-54	(UA)	3M™ Півмаска багаторазового використання	85-88
(IT) (CH)	Semimaschera riutilizzabile 3M™	14-17	(CZ)	3M™ Polomaska pro opakované použití	55-58	(HR)	3M™ Polumaska za višekratnu upotrebu	89-91
(ES)	3M™ Media Máscara Reutilizable	18-21	(SK)	3M™ Polomaska na viacnásobné použitie	59-61	(BG)	3M™ Полумаска за многократна употреба	92-95
(NL) (BE)	3M™ Herbruikbaar Halfgelaatsmasker	22-25	(SI)	3M™ Polmaska za večkratno uporabo	62-64	(RS)	3M™ Višekratne Polumaskе	96-98
(SE)	3M™ Återanvändningsbar halvmask	26-28	(IL)	3M™ מסכת חצי פנים לימוש חוזר	65-67	(TR)	3M™ Tekrar Kullanılabilir Yarım Maskeler	99-102
(DK)	3M™ halvmaske til genbrug	29-31	(EE)	3M™ Korduvkasutatav poolmask	68-70	(KZ)	3M™ Бірнеше рет пайдаланатын жартылай бетперде	103-105
(NO)	3M™ Halvmaske for gjenbruk	32-34	(LV)	3M™ atkārtoti lietojama pusmaska	71-73	(AU) (NZ)	3M™ Re-usable Half Mask	106-108
(FI)	3M™ uudelleenkäytettävä puolinaamari	35-38	(LT)	3M™ Daugkartinio naudojimo puskaukė	74-76	(AE)	3M™ قناع نصفی يُعاد استخدامه	109-111
(PT)	3M™ Meia máscara reutilizável	39-42						

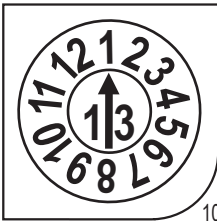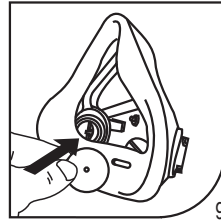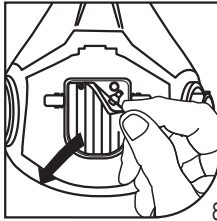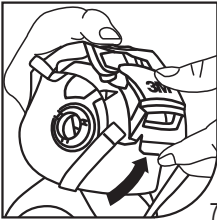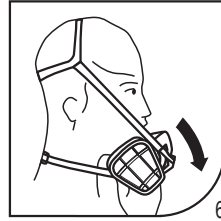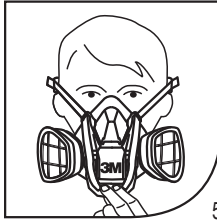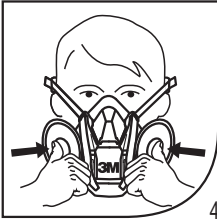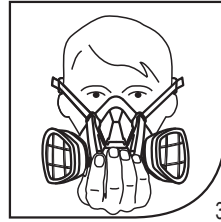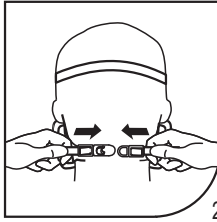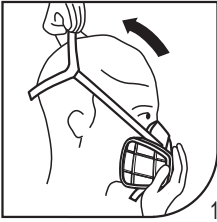Please read these instructions in conjunction with the appropriate 3M™ Filter Leaflet and with the 3M™ Face Piece Reference Leaflet where you will find information on:

- Approved combinations of 3M™ filters
- Accessories
- Spare parts

## SYSTEM DESCRIPTION

These products meet the requirements of EN 140:1998, re-usable half masks and should be used in combination with a pair of approved 3M filters (see Reference Leaflet) to form a filtering apparatus for respiratory protection. Such an apparatus is designed to remove potentially harmful gases, vapours and / or particulates from the surrounding atmosphere.

## ⚠ WARNINGS AND LIMITATIONS

Particular attention should be given to warning statements where indicated.

## ⚠ WARNINGS

Always be sure that the complete product is:

- Suitable for the application;
- Fitted correctly;
- Worn during all periods of exposure;
- Replaced when necessary.

**⚠ Proper selection, training, use and appropriate maintenance are essential in order for the product to help protect the wearer from certain airborne contaminants. Failure to follow all instructions on the use of these respiratory protection products and/or failure to properly wear the complete product during all periods of exposure may adversely affect the wearer's health, lead to severe or life threatening illness or permanent disability.**

- For suitability and proper use follow local regulations and refer to all information supplied or contact a safety professional or 3M
- Only use with the 3M™ Filters and spares and accessories listed and within the usage conditions given in the Technical Specification.
- Never modify or alter this product. Replace parts only with original 3M spare parts.
- Maintenance, servicing and repair must only be carried out by properly trained personnel.
- General inspection must be carried out before use and monthly. See Inspection procedures for specific details.
- Do not use in atmospheres containing less than 19.5% oxygen. (3M definition. Individual countries

may apply their own limits on oxygen deficiency. Seek advice if in doubt).

- Do not use these products in oxygen or oxygen-enriched atmospheres.
- Do not use for respiratory protection against atmospheric contaminants/concentrations which have poor warning properties or are unknown or immediately dangerous to life and health (IDLH) or against contaminants/concentrations which generate high heats of reaction with chemical filters.
- Select and wear appropriate protection when exposed to sparks and/or flames.
- In case of intended use in explosive atmospheres, contact 3M Technical Service.

**⚠ Do not use with beards or other facial hair that may inhibit contact between the face and the product thus preventing a good seal.**

- Leave the contaminated area immediately if:  
a) Breathing becomes difficult or increased breathing resistance occurs. b) Dizziness or other distress occurs. c) You smell or taste contaminants or irritation occurs. d) Airflow to the facepiece decreases or stops. e) Any part of the system becomes damaged.
- National regulations may impose specific limitations on the use of filters depending on the filter class and the facepiece used. Use of any 3M™ Facepiece / Filter combination should be in accordance with applicable Health and Safety standards, Respiratory Selection Tables or in accordance with the recommendations of an Occupational Hygienist.
- Materials which may come into contact with the wearer's skin are not known to cause allergic reactions to the majority of individuals.
- These products do not contain components made from natural rubber latex.

## PREPARATION FOR USE UNPACKING

Inspect the package contents for shipping damage and ensure all components are present.

Check apparatus is complete, undamaged and correctly assembled The "QL" mark on the latch identifies this facepiece as a 3M™ Half Mask 6500QL Series.

### Inspection

The following inspection procedure is recommended before use:

1. Check the facepiece for cracks, tears and dirt. Be certain the facepiece, especially the face seal area, is not distorted. The material must be pliable – not stiff.
2. Examine the inhalation valves for cracks or tears. Lift valves and inspect valve seat for dirt or cracks.
3. Ensure that the head straps are intact and have good elasticity.
4. Examine all plastic parts for signs of cracks

or fatigue. 5. Remove the exhalation valve cover and examine the exhalation valve and valve seat for signs of dirt, distortion, cracks or tears. Replace the exhalation valve cover.

## ASSEMBLY INSTRUCTIONS

See appropriate product user instruction (eg. 3M filter leaflet)

## FITTING INSTRUCTIONS

**⚠ Before initial donning, if using 6500QL (Quick Latch) product, ensure that the quick latch is closed and the upper straps are seated in the latch guides.**

1. Adjust head cradle size as needed to fit comfortably on head. 2. Place the respirator over the nose and mouth, fitting it comfortably on the bridge of the nose, then pull the head harness over the crown of the head. Figure 1. 3. Take a bottom strap in each hand, place the straps at the back of the neck and hook the straps together. See Fig. 2. 4. Tighten the top head harness first by pulling on the ends of the straps to achieve a comfortable and secure fit. Tighten the bottom straps in a similar manner. (Strap tension may be decreased by pushing out on the back of the buckles)

**IMPORTANT NOTICE: After initial donning of the 6500QL Quick Latch Half Mask, once straps are adjusted to proper length, subsequent donning can be done with quick latch initially in open or closed position. To don with the quick latch in the open position, follow steps as above for fitting the head harness, then raise mask to the in-use position according to the procedure described in the "Using Quick Latch Drop-Down Suspension" section of these User Instructions.** 5. Perform a positive and/or negative pressure face fit check.

**These fitting instructions MUST be followed each time respirator is worn.**

## FIT CHECK

• **Positive Pressure User Seal Check (for all approved configurations)** (all Filters except 3M™ 6035 /6038/ 2000 Series Filters).

1. Place the palm of your hand over the exhalation valve and exhale gently. See Fig. 3. If the facepiece bulges slightly and no air leaks are detected between the face and the facepiece, a proper seal has been obtained.

If face seal air leakage is detected, reposition the respirator on your face and/or readjust the tension of the straps to eliminate the leakage and recheck seal.

• **Negative Pressure User Seal Check with 6035 / 6038 Filters Also 2000 Series filters.**

Press your thumbs into the central indentation of the filters (2000 Series), or press filter cover and filter body together (6035/38), inhale gently and hold your breath

for five to ten seconds.

If the facepiece collapses slightly a proper fit has been achieved. See Fig. 4. If face seal air leakage is detected, reposition the respirator on your face and/or readjust the tension of the straps to eliminate the leakage and recheck seal.

**⚠ If you CANNOT achieve a proper fit DO NOT enter the hazardous area.**

### Quantitative Fit Testing

Users should be fit tested in accordance with national requirements.

For information regarding fit testing procedures, please contact 3M.

## USING QUICK LATCH DROP-DOWN SUSPENSION

**IMPORTANT NOTICE:** 6500QL Quick Latch Half Mask should only be moved to drop-down position in non-contaminated environments. The Quick Latch enables wearers to don and doff the half mask quickly without moving the head harness from its position on the crown of the head.

Follow these steps to lower the half mask from the in-use position on the face to the drop-down position below the chin.

1. Grasp the bottom of the quick latch, at chin level, with the thumb and first two fingers. (Fig.5) 2. Pull up and open the quick latch, as the half mask loosens from the face and begins to slide to the drop-down position. The Half Mask will rest below the chin (Fig. 6).

Follow these steps to raise the half mask from the drop-down position below the chin to the in-use position on the face.

1. Grasp the front of the half mask with the thumb at the bottom of the half mask and the first two fingers at the top of the opened quick latch. 2. Place the half mask over the mouth and nose 3. Push the quick latch down and click into the lower, closed position

## DOFFING

**⚠ Do not remove the face piece, or filters until you have vacated the contaminated area.**

1. Release the tension on the head straps. 2. Carefully lift facepiece off the face and remove the respirator by lifting up and away from the face.

### Replacement Parts Instructions

3M™ Rugged Comfort Half Mask 6500 and 6500QL consist of identical parts, with the exception of the Head Harness Assembly & Valve Cover components.

### 3M™ Head Harness Assembly & Valve Cover Replacement

1. To remove the Quicklatch suspension head harness assembly & valve cover, first open the quick latch. For both quicklatch and standard assemblies, place thumb beneath valve cover and fingers at the top. Pull the

valve cover bottom away from facepiece with the thumb and then up to release the valve cover (Fig 7).

2. Replace Head Harness Assembly & Valve Cover by positioning the peg on the facepiece into the notch at the top of the valve cover. Press the bottom of the valve cover into the facepiece. When properly engaged, an audible snap should be heard.

#### Exhalation Valve Replacement

1. Remove 3M™ Head Harness Assembly & Valve Cover 6581(Std)/6582 (QL) by pulling away from facepiece (Fig. 7). 2. Remove 3M™ Exhalation Valve 6583 from valve seat by grasping the valve and pulling each valve stem out from the valve seat (Fig. 8). 3. Inspect valve seat making certain it is clean and in good condition. 4. Replace 3M™ Exhalation Valve 6583 by inserting stems into the two holes until they are visible from inside the facepiece and then pulling the stems through until they are seated and secured. 5. Replace 3M™ Head Harness Assembly & Valve as described previously.

**NOTE: Conduct a negative pressure user seal check to ensure exhalation valve is functioning properly.**

#### Inhalation Valve Replacement

1. Remove existing valve(s) by grasping valve and pulling valve off the centre post. 2. Replace valve(s) by pressing the hole in the valve over the centre post on the valve seat (Fig. 9). Be certain valve(s) is fully engaged through valve post(s) and lays flat.

## CLEANING AND DISINFECTION

Cleaning is recommended after each use.

Use the 3M™ 105 Face Seal Cleaner to clean the respirator.

Remove filters. The exhalation valve cover, exhalation valve and inhalation valves can also be disassembled if necessary

Clean parts (excluding filters) by immersing in warm cleaning solution (water temperature not to exceed 50°C), scrub with soft brush until clean. Add neutral detergent if necessary. Disinfect respirator by soaking in a solution of quaternary ammonia disinfectant or sodium hypochlorite or other disinfectant. Rinse in clean, warm water and air dry at room temperature in a non-contaminated atmosphere. Do not reassemble parts until facepiece is completely dry.

**CAUTION** Do not use cleaners containing lanolin or other oils on these products.

## STORAGE AND TRANSPORTATION

These products should be stored in the packaging provided in dry, clean conditions away from sources of high temperature and petrol and solvent vapours. The original packaging is suitable for transporting the product throughout the European Union. When stored as stated, the expected shelf life of the product is 5 years from date of manufacture. Store in accordance

with manufacturer's instructions, see packaging.

↓ End of Shelf Life ↓ Temperature Range

☂ Maximum Relative Humidity

🏠 Name and address of Manufacturer

Dispose in accordance with local regulations

The date of manufacture can be established by examining the date clock on the inside of the facepiece. See Fig. 10. The inner segment will display the year, and the arrow will point to the month of manufacture in the outer segment. The example shows 01/01/2013 - 31/01/2013. The head harness assembly and filter holder are also marked with a date wheel indicating their month and year of manufacture.

## TECHNICAL SPECIFICATION

Respiratory Protection EN140 - for use with 3M approved bayonet filters to EN14387 or EN143 as detailed in the Reference Leaflet.

When used with the following 3M equipment, this respirator can be used in concentrations of contaminant up to:

3M™ Re-usable Half Mask with	Maximum Use Concentration
P1 Particulate Filters	4 x WEL
P2 Particulate Filters	10 x WEL
P3 Particulate Filters	20 x WEL
Class 1 Gas & Vapour Filters	10 x WEL or 1000 ppm (0.1% vol) whichever is lower
Class 2 Gas & Vapour Filters	10 x WEL or 5000 ppm (0.5% vol) whichever is lower
Special / Combination Filters	Contact 3M for additional information.

WEL = Workplace Exposure Limit  
Assigned Protection Factors are taken from HSE publications.

Maximum Operating Temperature: +50 °C. Care should be taken when using the equipment at low temperatures as excessive moisture may cause the valves to freeze.

## APPROVALS

These products meet the requirements of the European Community Directive 89/686/EEC (Personal Protective Equipment Directive) and are thus CE marked. Certification under Article 10, EC Type-Examination and Article 11, EC Quality Control, has been issued by BSI, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP, UK (Notified body number 0086). 3M United Kingdom PLC, 3M Centre, Cain Road, Bracknell, Berkshire, RG12 8HT, UK.

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## SYSTEM DESCRIPTION

These products meet the requirements of EN 140:1998, re-usable half masks and should be used in combination with a pair of approved 3M filters (see Reference Leaflet) to form a filtering apparatus for respiratory protection. Such apparatus is designed to remove potentially harmful particulates from the surrounding atmosphere.

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** Proper selection, training, use and appropriate maintenance are essential in order for the product to help protect the wearer from certain airborne contaminants. Failure to follow all instructions on the use of these respiratory protection products and/or failure to properly wear the complete product during all periods of exposure may adversely affect the wearer's health, lead to severe or life threatening illness or permanent disability.**

- For suitability and proper use follow local regulations, refer to all information supplied or contact an occupational hygienist, safety professional or 3M representative on the Tech Assist Helpline 3M Australia 1800 024 464, 3M New Zealand 0800 364 357.
- Only use with the 3M™ Filters and spares and accessories listed and within the usage conditions given in the Technical Specification.
- Never modify or alter this product. Replace parts only with original 3M spare parts.
- Maintenance, servicing and repair must only be carried out by properly trained personnel.
- General inspection must be carried out before use or monthly if not in regular use. See Inspection procedures for specific details.

- Do not use in atmospheres containing less than 19.5% oxygen. (3M definition. Individual countries may apply their own limits on oxygen deficiency. Seek advice if in doubt).
- Do not use these products in oxygen or oxygen-enriched atmospheres.
- Do not use for respiratory protection against atmospheric contaminants/concentrations which have poor warning properties or are unknown or immediately dangerous to life and health (IDLH) or against contaminants/concentrations which generate high heats of reaction with chemical filters.
- Select and wear appropriate protection when exposed to sparks and/or flames.
- In case of intended use in explosive atmospheres, contact 3M.

** Do not use with beards or other facial hair that may inhibit contact between the face and the product thus preventing a good seal.**

- Leave the contaminated area immediately if:  
a) Breathing becomes difficult or increased breathing resistance occurs. b) Dizziness or other distress occurs. c) You smell or taste contaminants or irritation occurs. d) Airflow to the face piece decreases or stops. e) Any part of the system becomes damaged.
- National regulations may impose specific limitations on the use of filters depending on the filter class and the facepiece used. Use of any 3M™ Face piece / Filter combination should be in accordance with applicable Health and Safety standards, AS/NZS 1715, Respiratory Selection Tables or in accordance with the recommendations of an Occupational Hygienist.
- Materials which may come into contact with the wearer's skin are not known to cause allergic reactions to the majority of individuals.
- These products do not contain components made from natural rubber latex.

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Check apparatus is complete, undamaged and correctly assembled. The "QL" mark on the latch identifies this facepiece as a 3M™ Half Mask 6500QLSeries.

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1. Check the face piece for cracks, tears and dirt. Be certain the face piece, especially the face seal area, is not distorted. The material must be pliable – not stiff.
2. Examine the inhalation valves for cracks or tears. Lift valves and inspect valve seat for dirt or cracks.

3. Ensure that the head straps are intact and have good elasticity. 4. Examine all plastic parts for signs of cracks or fatigue. 5. Remove the exhalation valve cover and examine the exhalation valve and valve seat for signs of dirt, distortion, cracks or tears. Replace the exhalation valve cover.

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## FITTING INSTRUCTIONS

**△ Before initial donning, if using 6500QL (Quick latch) product, ensure that the quick latch is closed and the upper straps are seated in the latch guides.**

1. Adjust head cradle size as needed to fit comfortably on head. 2. Place the respirator over the nose and mouth, fitting it comfortably on the bridge of the nose, then pull the head harness over the crown of the head. Figure 1, 3. Take a bottom strap in each hand, place the straps at the back of the neck and hook the straps together. See Fig. 2, 4. Tighten the top head harness first by pulling on the ends of the straps to achieve a comfortable and secure fit. Tighten the bottom straps in a similar manner. (Strap tension may be decreased by pushing out on the back side of the buckles)

**IMPORTANT NOTICE: After initial donning of the 6500QL Quick Latch Half Mask, once straps are adjusted to proper length, subsequent donning can be done with quick latch initially in open or closed position. To don with the quick latch in the open position, follow steps as above for fitting the head harness, then raise mask to the in-use position according to the procedure described in the "Using Quick Latch Drop-Down Suspension" section of these User Instructions. 5. Perform a positive and/or negative pressure face fit check.**

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If faceseal air leakage is detected, reposition the respirator on your face and/or readjust the tension of the straps to eliminate the leakage and recheck seal.

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Press your thumbs into the central indentation of the

filters (2000 Series), or press filter cover and filter body together (6035/6038), inhale gently and hold your breath for ten seconds

If the face piece collapses slightly a proper fit has been achieved. See Fig. 4. If faceseal air leakage is detected, reposition the respirator on your face and/or readjust the tension of the straps to eliminate the leakage and recheck seal.

**△ If you CANNOT achieve a proper fit DO NOT enter the hazardous area.**

### Quantitative Fit Testing

It is recommended that wearers should be fit tested in accordance with AS/NZS 1715.

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1. Grasp the bottom of the quick latch, at chin level, with the thumb and first two fingers. (Fig. 5). 2. Pull up and open the quick latch, as the half mask loosens from the face and begins to slide to the drop-down position. Half Mask will rest below the chin (Fig. 6).

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1. Grasp the front of the half mask with the thumb at the bottom of the half mask and the first two fingers at the top of the opened quick latch. 2. Place the half mask over the mouth and nose. 3. Push the quick latch down and click into the lower, closed position.

## DOFFING

**△ Do not remove the face piece, or filters until you have vacated the contaminated area.**

1. Release the tension on the head straps. 2. Carefully lift face piece off the face and remove the respirator by lifting up and away from the face.

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### 3M™ Head Harness Assembly & Valve Cover

#### Replacement

1. To remove the Quicklatch suspension head harness assembly & valve cover, first open the quick latch. For both quicklatch and standard assemblies, place thumb

beneath valve cover and fingers at the top. Pull the valve cover bottom away from facepiece with the thumb and then up to release the valve cover (Fig 7).

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To clean the respirator, the 3M™ 504 Wipe should be used to wipe the face seal of the product.

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## STORAGE AND TRANSPORTATION

These products should be stored in the packaging provided in dry, clean conditions away from sources of high temperature and petrol and solvent vapours. The original packaging is suitable for transporting the

product. When stored as stated, the expected shelf life of the 605X, 6075 and 609X products is 5 years from date of manufacture. Store in accordance with manufacturer's instructions, see packaging.

 End of Shelf Life |  Temperature Range

 Maximum Relative Humidity

 Name and address of Manufacturer

 Dispose in accordance with local regulations

The date of manufacture can be established by examining the date clock on the inside of the facepiece. See Fig. 10. The inner segment will display the year, and the arrow will point to the month of manufacture in the outer segment. The example shows 01/01/2013 - 31/01/2013. The head harness assembly and filter holder are also marked with a date wheel indicating their month and year of manufacture

## TECHNICAL SPECIFICATION

Respiratory Protection From AS/NZS 1715 - for use with 3M approved bayonet filters to AS/NZS 1716.

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3M™ Re-usable Half Mask with	Maximum Exposure Limitations
P1 Particulate Filters	10 x ES
P2 Particulate Filters	10 x ES*
P3 Particulate Filters	50 x ES*
Class 1 Gas & Vapour Filters	10 x ES* or 1000 ppm (0.1% vol) whichever is lower
Class 2 Gas & Vapour Filters	10 x ES* or 1000 ppm (whichever is lower)
Special / Combination Filters	Contact 3M for additional information.

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## APPROVALS

These products meet the requirements of the European Community Directive 89/686/EEC (Personal Protective Equipment Directive) and are thus CE marked. Certification under Article 10, EC Type-Examination and Article 11, EC Quality Control, has been issued by BSI, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP; UK (Notified body number 0086). 3M United Kingdom PLC, 3M Centre, Cain Road, Bracknell, Berkshire, RG12 8HT, UK.

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**3M Australia Pty Limited**
**3M New Zealand Limited**
**3M Gulf Ltd.**
**3M Egypt Trading Ltd.**